



ILS Scooter Trailer Assembly Guide

1



Separate wheels and box frame

2



Place wheels onto the centre axle pole and secure in position with lever

3



Place U shaped bracket over the bottom hole that attaches to the frame and place bolt through the hole

4



With the U-shaped bracket attached install attach a bolt through the bottom pole of the front of the trailer frame, also insert bolt through the top pole

5



Tighten firmly securing the pole at two points to the trailer frame

6



Place the box inside the trailer

7



Place trailer cover over the top of the box
Using the Velcro secure that cover to the trailer frame

8



Install four reflectors onto the trailer frame with screw driver provided (Each corner)

9



Use long bolt, washer and nut for attachment of stabilizer pole

10



Attach stabilizer bar to the U-shaped bracket at the front of the trailer

11



Attach locking clip to secure stabilizer bar in position

13



Use chrome hitch with two bolts and washers

14



Attach chrome hitch to the end of the L-shaped pole and secure with supplied wrench

15



Using two metal pins attach hook and receiver attachment to the trailer hitch as shown in photo

16



Please see completed model