

cura¹

2702 - 3 Channel Cordless Falls Monitor



What's in the box?



2703 Wireless Output Falls Monitor
2742 Soft Monitor Cover
2699 Fixing Strap
3504 Mounting Bracket
3 X AA Batteries
QuickStart User Guide

Compatible & Optional Accessories

2719 & 2720 Cordless Pad / Mat Transmitters
& all Pads & Mats that use them

2721 iBeam, 2406 Plug Pack 6 Vdc, 2599 Wireless SOS Call Button,
2588 Personal Emergency Transmitter (PET), 3502 Wireless Handset,
3503 Wireless Door Switch, 2558 Wireless Waterproof Call Button with
Pull Cord, 3504 ActiveCare Bracket

Nurse Call Cables: 2643, 2646, 2493, 2461, 2460

cura¹


10 Gray Street, Kilkenny South Australia 5009
www.cura1.com

1300-12-CARE (2273)

UM-048a



All our products are supplied with a full warranty and are guaranteed to be free from defects in material and workmanship.

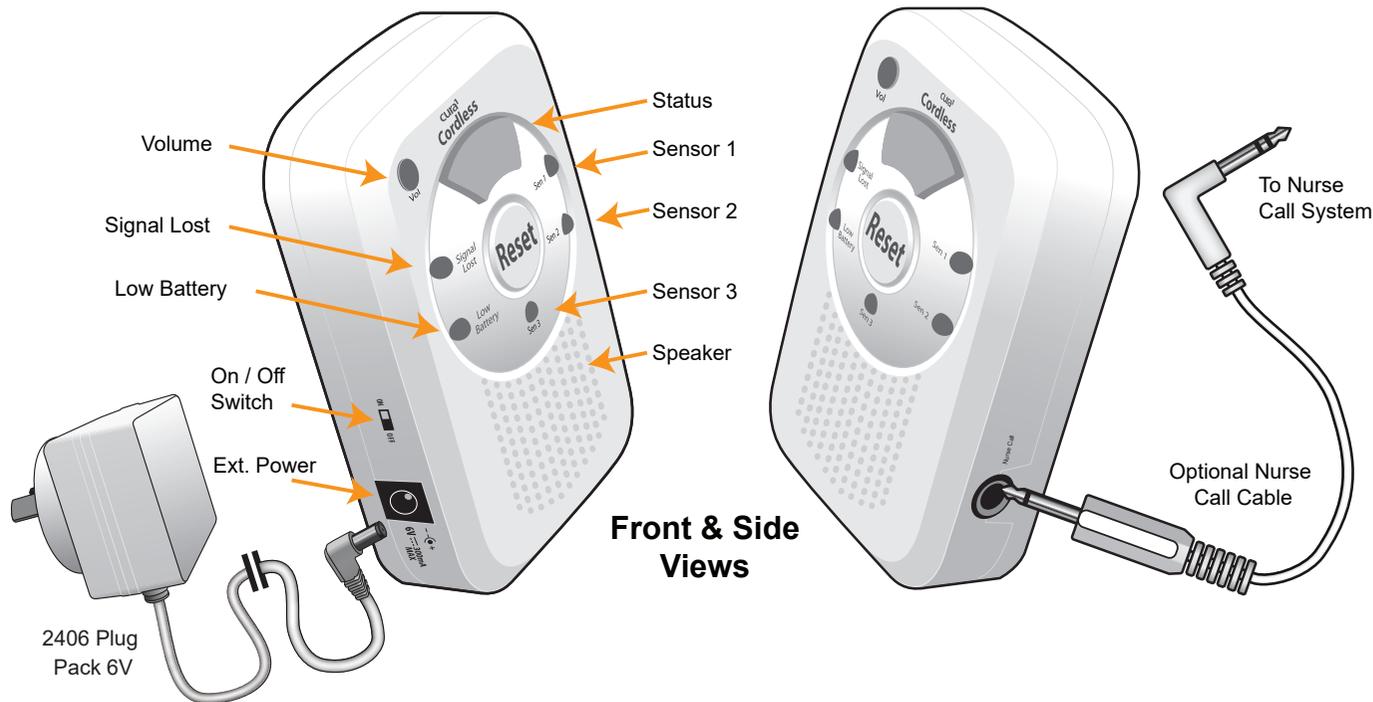

OPERATION

Volume Adjustment

Press the **Vol** button to cycle through 3 alarm volume levels. If a nurse call cable is plugged in then there are 4 volume levels to choose from with the 4th being silent.

Sensor Alarms

In an alarm state the **Status** light and the LED that is associated with the alarming sensor (**Sen 1**, **Sen 2** or **Sen 3**) will flash. The audio alarm will sound unless a nurse call cable is plugged in and the volume is set to silent.

Clearing a Sensor Alarm


Press the **Reset** button.

Signal Lost

When a paired Pad or Mat is out of range, turned off or otherwise disabled, the **Signal Lost** LED and the sensor LED (**Sen 1**, **Sen 2** or **Sen 3**) associated with that sensor, will flash.

Low Battery

When the **Low Bat** LED blinks, its time to replace the batteries. If a paired Pad or Mat has a low battery then the sensor LED (**Sen 1**, **Sen 2** or **Sen 3**) associated with that sensor will also flash.

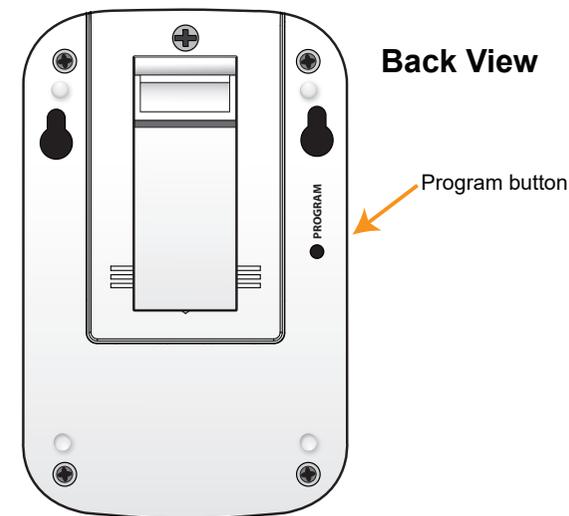

MORE INFORMATION

For detailed information of all 2702 Monitor functions please refer to the full technical manual for this product or contact your cura¹ distributor.

QUICK START GUIDE

2702 - 3 Channel Cordless Falls Monitor quick setup for use with cura¹ cordless pads, mats, buttons and sensors.


Step 1

Insert 3 x AA Alkaline Batteries

The 2702 3 Channel Cordless Falls Monitor is designed to work with alkaline batteries equivalent to those originally supplied with the device.

Operable alkaline batteries must be installed in the device at all times during operation. Batteries must be removed from the device while it's in storage.

Use screwdriver, (accepts both Phillips or flat head) to undo screw & slide battery cover UP to open.

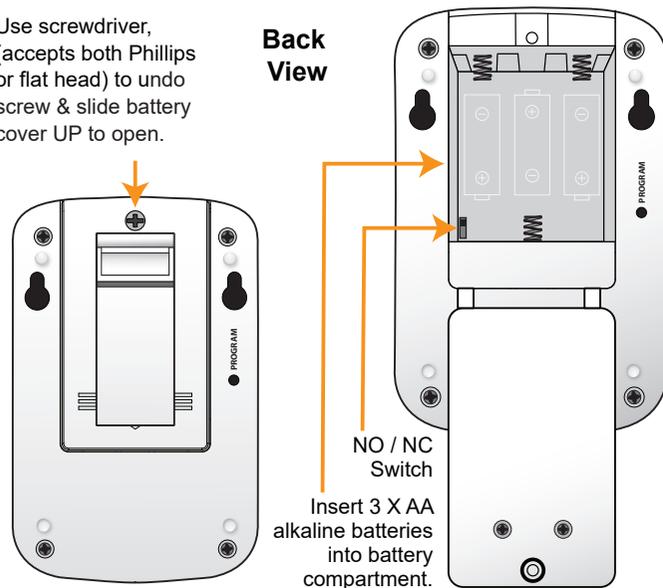

Step 2

Collect devices for pairing

Gather up to 3 compatible cura¹ cordless devices that you would like the 2702 to monitor and have them within easy reach, eg. bed / chair pads, call buttons & movement sensors.

Step 3


Clear Monitor Memory

If your 2702 Monitor is brand new & never used then you can skip this step but if the Monitor has been used previously then it will be necessary to remove past pairings from its memory.

1. Move the **ON / OFF** switch of the 2702 to the **ON** position.
2. Press **PROGRAM** button to enter pairing.
3. Hold **Reset** button 5 for seconds to clear all paired devices.

Step 4

Pair with Cordless Pad/s or Floor Mat

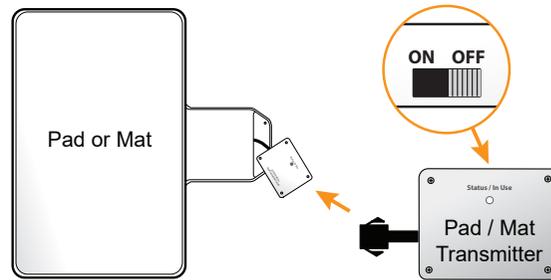

1. Each Pad or Mat must have a Cordless Transmitter that is connected & turned **OFF**.
2. Ensure that the **ON / OFF** switch of the 2702 Monitor is in the **ON** position.
3. Press & hold the **PROGRAM** button on the 2702 Monitor until you hear 3 beeps, indicating Pairing Mode is enabled.
4. Turn **ON** the Pad / Mat Transmitter that you would like to pair with the 1st memory slot (**Sen 1**) of the 2702 Monitor.

After a successful pairing the Monitor will automatically exit pairing mode and beep 3 times. The monitor will also exit pairing mode after 1 minute of inactivity.

5. Repeat points 3 and 4 to pair additional Pad / Mat Transmitters with **Sen 2** and **Sen 3** memory slots.

Step 5


Pair with other transmitters or sensors

Pair the 2702 Monitor with non-Pad / Mat transmitters such as a call button or motion detector.

1. Press & hold the **PROGRAM** button on the 2702 Monitor until you hear 3 beeps, indicating pairing mode is enabled.
2. Trigger the transmitter or sensor that you want to pair. After a successful pairing the Monitor will automatically exit pairing mode and beep 3 times.

Step 6

Plug into nurse call system (if required) and test all paired devices in their final locations.

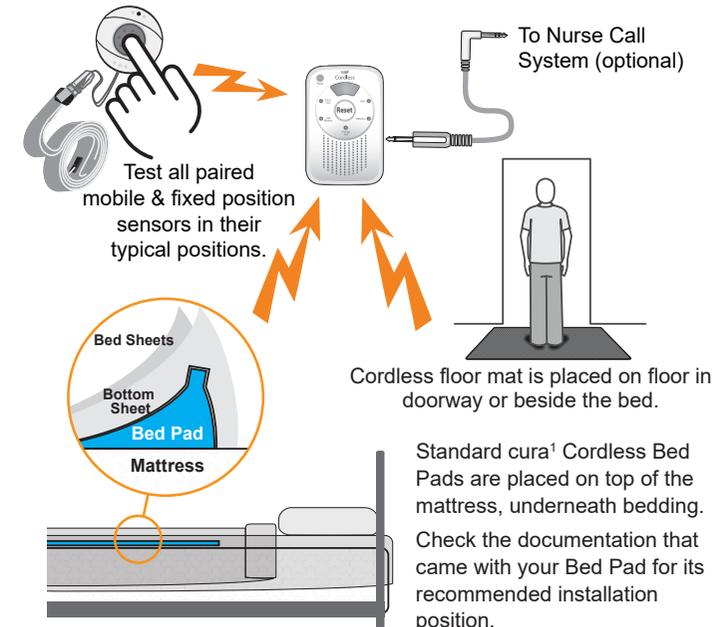

Test falls monitor prior to every use.