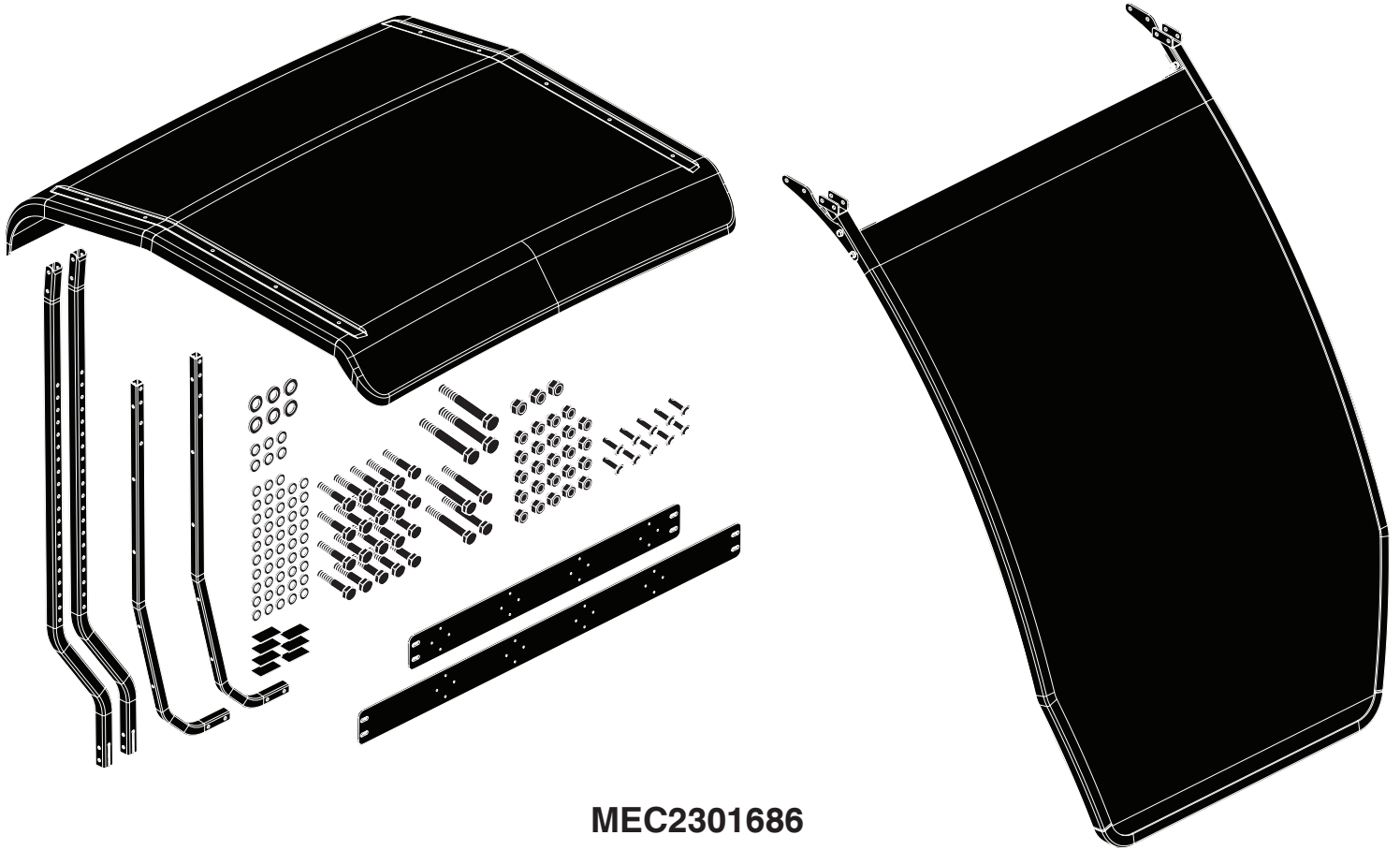
**Main Canopy Installation • Installation de l'auvent principal
Installatie van de hoofdluifel • Installation des Hauptdachs
Installazione della tettoia principale • Instalación del dosel principal**

MEC2301686

If you require any assistance with these instructions, please call **Pride Technical Service** at **1-800-800-8586**.
Canada providers, phone **1-888-570-1113**.

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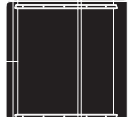
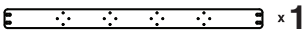
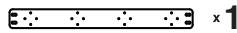
Tools needed:

- 10mm socket or wrench
- 13mm socket or wrench
- 4mm hex key

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To Prepare Scooter for Canopy Installation:

Ensure the scooter is turned off and in drive mode. Refer to your owner's manual. If not already completed, install canopy mounting brackets to the scooter as per the applicable instructions.

WARNING! Ensure the scooter is placed on a hard, level surface before proceeding.

CAUTION! It is recommended that two people perform this installation.

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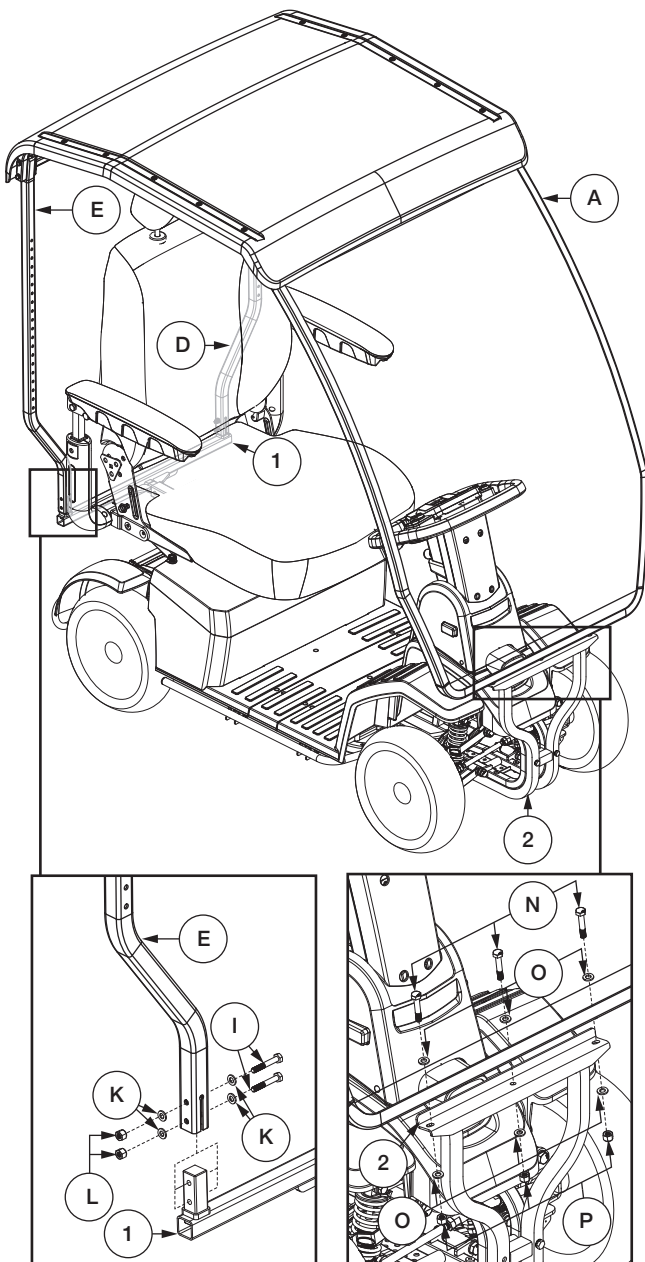
To Mount the Canopy:

With the assistance of another person, lift the canopy assembly over the scooter and insert the Rear Mounting Bracket (1) into the Rear Upright Support receivers (D/E) as shown. Using a 10mm wrench, secure with four (4) 40mm Hex Bolts (I), eight (8) M6 Washers (K), and four (4) Nylock Nuts (L). Leave fasteners slightly loose for later adjustment.

Align the three (3) holes on the inside of the front of the Windshield Assembly (A) with the holes on the front mounting bracket (2) as shown. Using a 13mm wrench, secure the canopy assembly to the front mounting bracket (2) with three (3) M8 Hex Bolts (N), six (6) M8 Washers (O), and three (3) M8 Nylock Nuts (P). Leave fasteners slightly loose for later adjustment.

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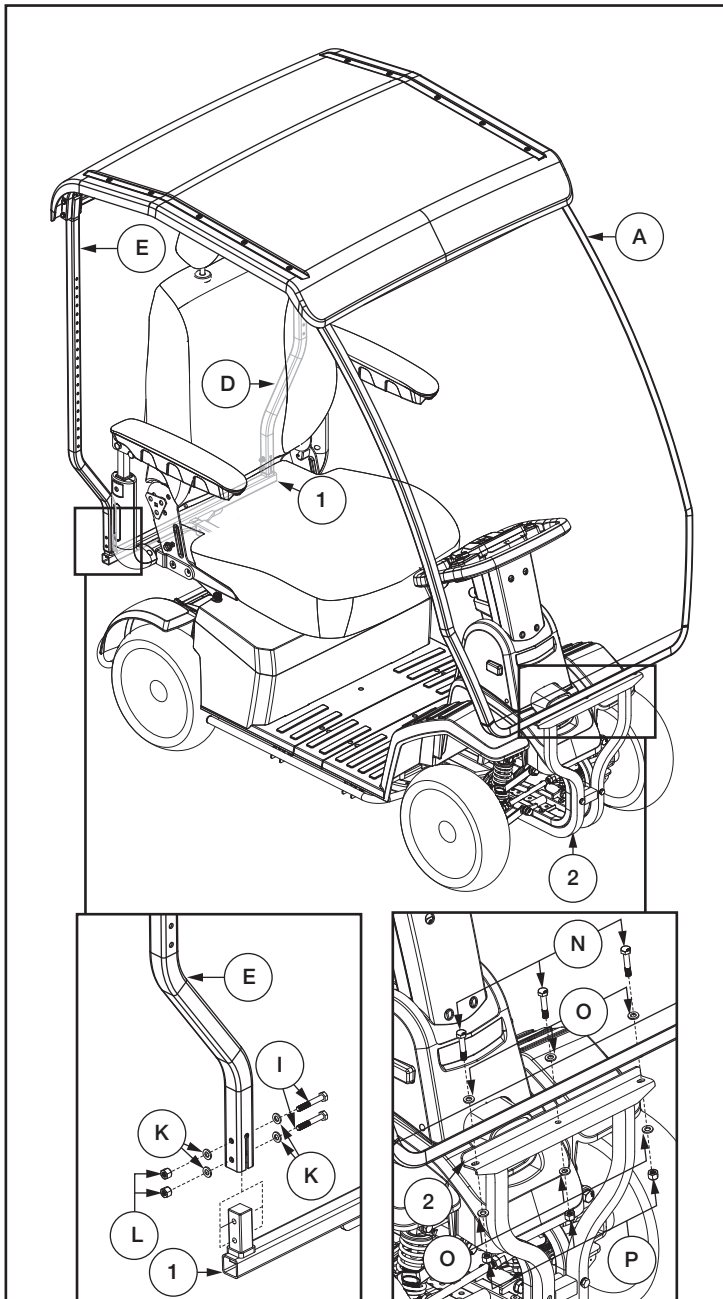
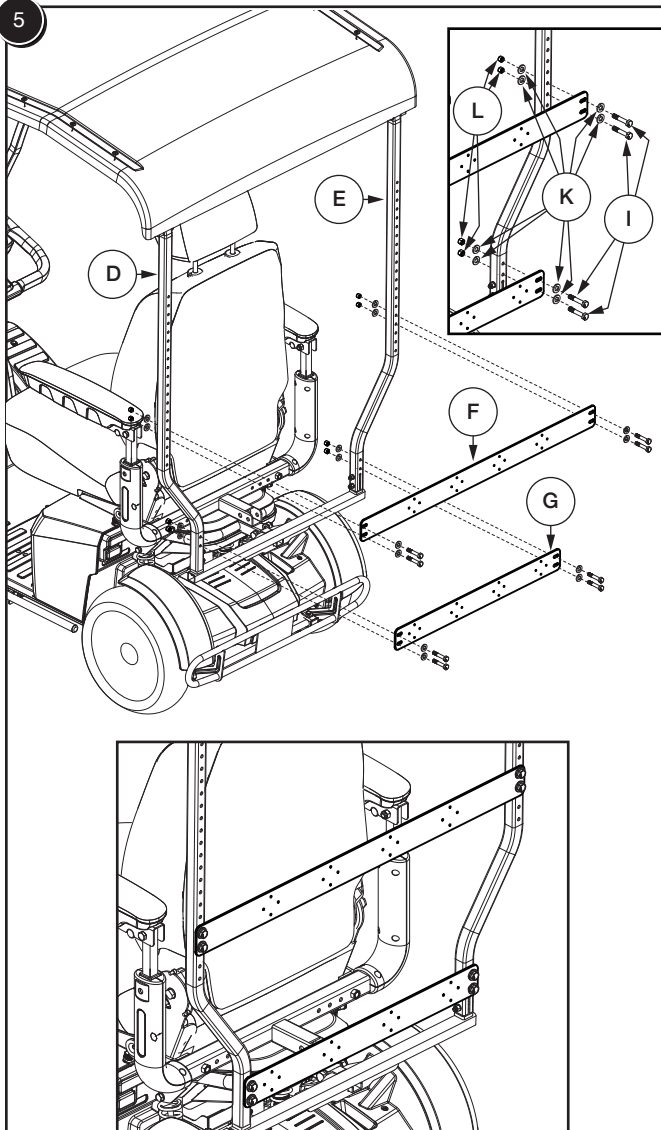
5

To Install the Accessory Mounting Plates:

Using a 10mm wrench, attach the upper and lower Accessory Mounting Plates (F, G) to the back of the Rear Upright Supports (D, E) as shown. Secure using eight (8) 40mm Hex Bolts (I), sixteen (16) M6 Washers (K), and eight (8) M6 Nylock Nuts on each side (L).

With all hardware in place, make any final alignment adjustments and fully tighten all loose fasteners.

NOTE: Remove the protective plastic cover from the windshield before operating.

WARNING! The weight of the canopy must be included when considering your scooter's overall weight capacity.

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