



AIR CUSHION AND PUMP SYSTEM (EP-C)

Quick Guide


1. POWER UP

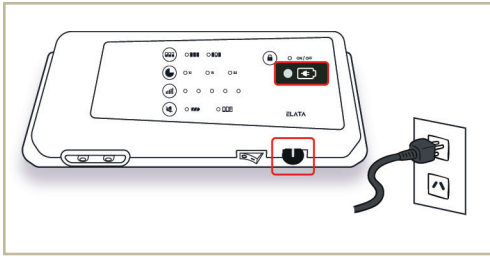

Connect the power cable to pump & wall outlet. Charge until battery light turns off

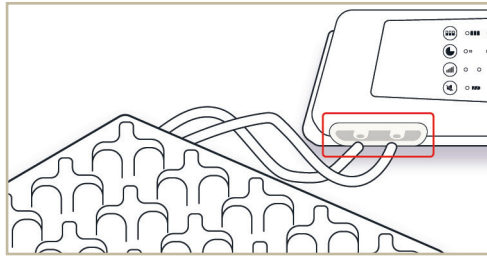

Connect the pump to cushion tubes

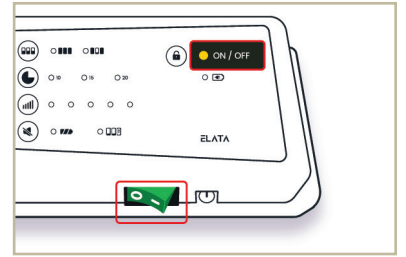

Press Power Switch. The On/Off indicator should light yellow

2. SELECT SETTINGS



If Alternating mode, is selected you can set the cycle time. cycle time selected will light up green




Select the Pressure level between 1 - 5 based on the approximate user weight



Pressure Setting	1	2	3	4	5
Approx. User Weight	60kg	80kg	100kg	160kg	220kg

It is recommended that the user starts at the pressure setting closest to their recommended weight and then adjust it lower or higher based on their comfort

3. START THERAPY

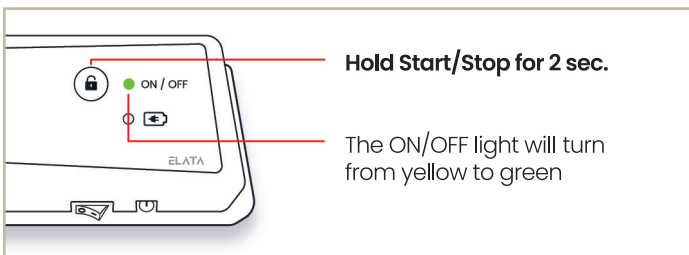

Adjust mode, time, or pressure at any time based on comfort

TROUBLESHOOTING

If you hear a beep coming from your Elata Air Cushion System, it will indicate the following:

- ■■■ Indicates low battery warning
- ■■■ Indicates low pressure warning

Press Mute key  to silence alarms