



## Royale Multi-Motion Companion Bed Base

### User Guide

The Royale Multi-Motion Companion Bed Base is designed to provide a stable, non-adjustable sleeping surface for partners or caregivers. It is fully compatible with Royale Multi-Motion adjustable homecare beds, offering a practical and supportive solution for shared sleeping arrangements without compromising care or comfort.

#### Available Range:

Code	Description
S346406	Bed - Royale Multi-Motion Companion Base (Fixed Leg Frame) Long Single (Ardo Grey Base)
S346407	Bed - Royale Multi-Motion Companion Base (Fixed Leg Frame) Long Single (Jet Black Base)
S346408	Bed - Royale Multi-Motion Companion Base (Fixed Leg Frame) Long King Single (Ardo Grey Base)
S346409	Bed - Royale Multi-Motion Companion Base (Fixed Leg Frame) Long King Single (Jet Black Base)

---

#### Assembly Instructions

The bed base is supplied pre-assembled within the carton.

1. Carefully remove the bed base and components from the packaging.
2. Locate the six (6) legs provided.
3. Screw each leg into the designated receivers on the underside of the bed base.
4. Ensure all legs are securely tightened before use.

#### Mattress Retainer Bars:

- Four (4) mattress retainer bars are included.
- Insert one retainer bar into each corner of the mattress platform.
- These bars help keep the mattress securely in place during use.

---

#### Warranty

The Royale Multi-Motion Companion Bed Base is covered by a 3-year warranty from the date of purchase, covering defects in structural integrity under normal use.