

Configura[®] Comfort


Information & Assessment Form



USER GROUP

To provide comfortable seating to users with mild to moderate postural requirements, to help reduce the formation of pressure ulcers and help to stand or sit users with limited mobility.

APPROPRIATE FOR:

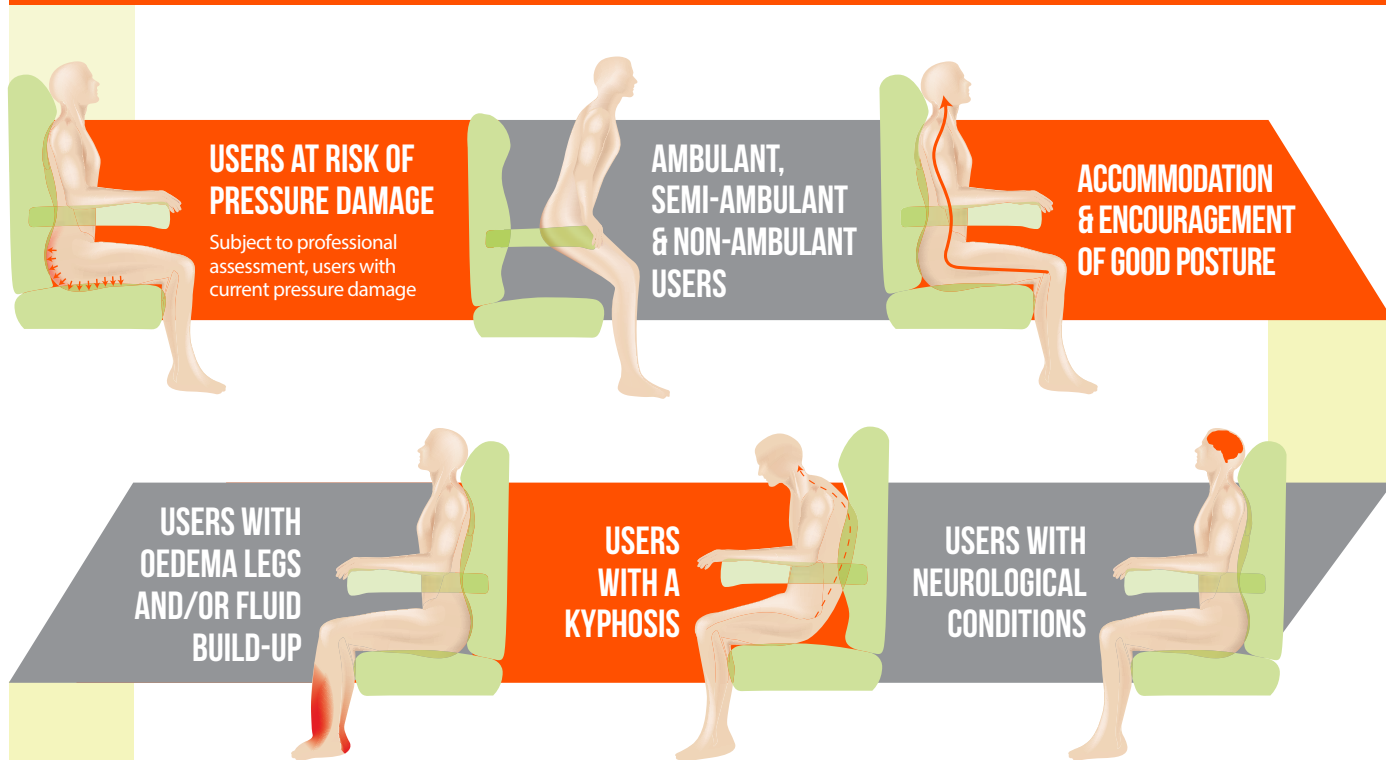

NOT APPROPRIATE FOR:

- 1** Demanding postural requirements
- 2** Users with fixtures and contractures
- 3** Users with a limited extension at the knees
- 4** Users who need to be transported from room to room
- 5** Bariatric users

INTENDED ENVIRONMENT



Domestic use



Care homes or nursing homes

STANDARD CHAIR SPECIFICATION

Configura® Comfort is the latest innovation from Accora and provides an unprecedented level of functionality and cost effectiveness.

Comfort is the result of acting on feedback collected from our customers over the past 5 years and it includes many features you have been asking for as standard.



'Reduce Shear' recline to maximise pelvic stability

Integrated self-supply overlay for easy integration of existing cushions*

Compatible with transfer aids including Cricket, ReTurn 7500 and Molift Raiser

Knock down design for easier storage and single person delivery

Pressure reducing fabric on backrest and armrest as standard

Colour matched vapour permeable fabric

Castors for easy manoeuvrability

More depth adjustment options using backrest brackets. Reduces the need for depth adjustment kits

60kg legrest capacity

Sliding armrest to facilitate transfer aids



**For further information on the integrated self-supply overlay please see page 11.*

PRESCRIBER DETAILS

Name:

Department:

Address:

.....

.....

Telephone:

Email:

USER DETAILS

Name:

Address:

.....

.....

Telephone:

Email:

User weight:

Pressure Care:

Posture:

Mobility:

Diagnosis:

Prognosis:

Transfer method:

Current seating:


Problems with current seating:

Notes:

.....

.....


.....

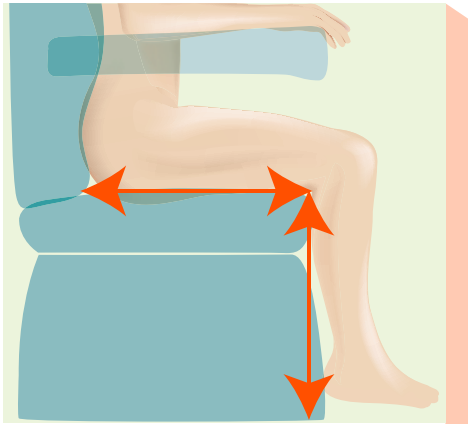


SIZE

Key thoughts:


- 3 measurements are required for selecting a chair; seat depth, seat height and seat width – see below drawings for clarity.
- When selecting a chair size, prioritise the chair measurements in the following order: seat depth, seat height then seat width. This will ensure the selected size is the right overall proportion for the user
- If you cannot find the size you need, check out the other chairs in our range or contact us as we may be able to supply a special size.

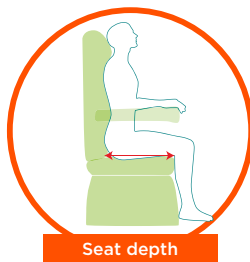

Seat depth

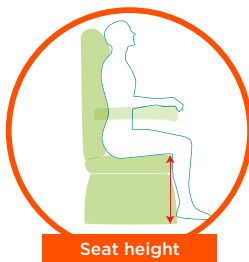


Seat height

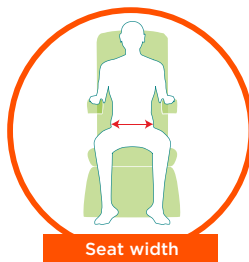

Seat width

Chair Size	Part Number	Seat Height	Seat Width	Seat Depth	Weight Capacity
SMALL	☐ CR5404 - DURATEK ☐ CR5422 - Beige Vinyl ☐ CR5443 - Black Vinyl	☐ 375 mm (15") ^{#5}	412 mm (16.5") ^{#6}	☐ 350 mm (14") ^{#1}	Seat: 160 kg Legrest: 60 kg
		☐ 400 mm (16")		☐ 400 mm (16") ^{#2}	
		☐ 425 mm (17") ^{#3}	450 mm (18")	☐ 450 mm (18")	
		☐ 450 mm (18") ^{#4}		☐ 500 mm (20")	
MEDIUM	☐ CR5424 - Beige Vinyl ☐ CR5444 - Black Vinyl	☐ 425 mm (17") ^{#5}	462 mm (18.5") ^{#6}	☐ 350 mm (14") ^{#1}	Seat: 160 kg Legrest: 60 kg
		☐ 450 mm (18")		☐ 400 mm (16") ^{#2}	
		☐ 475 mm (19") ^{#3}	500 mm (20")	☐ 450 mm (18")	
		☐ 500 mm (20") ^{#4}		☐ 500 mm (20")	
LARGE	☐ CR5426 - Beige Vinyl ☐ CR5445 - Black Vinyl	☐ 425 mm (17") ^{#5}	512 mm (20.5") ^{#6}	☐ 350 mm (14") ^{#1}	Seat: 160 kg Legrest: 60 kg
		☐ 450 mm (18")		☐ 400 mm (16") ^{#2}	
		☐ 475 mm (19") ^{#3}	550 mm (22")	☐ 450 mm (18")	
		☐ 500 mm (20") ^{#4}		☐ 500 mm (20")	
EXTRA LARGE	☐ CR5446 - Black Vinyl	☐ 375 mm (15") ^{#5}	562 mm (22.5") ^{#6}	☐ 350 mm (14") ^{#1}	Seat: 255 kg Legrest: 65 kg
		☐ 400 mm (16")		☐ 400 mm (16") ^{#2}	
		☐ 425 mm (17") ^{#3}	600 mm (24")	☐ 450 mm (18")	
		☐ 450 mm (18") ^{#4}		☐ 500 mm (20")	

#1 Depth Adjustment	#2 Depth Adjustment	#3 Height Adjustment	#4 Height Adjustment	#5 Height Reduction
☐ 2 x 2" Kits (100mm) CA2417 	☐ 2" Kit (50mm) CA2416 	☐ 1" Raising Kit (25mm) CA2421 	☐ 2" Raising Kit (50mm) CA2422 	☐ 1" Lowering Kit (18mm) CA2423 

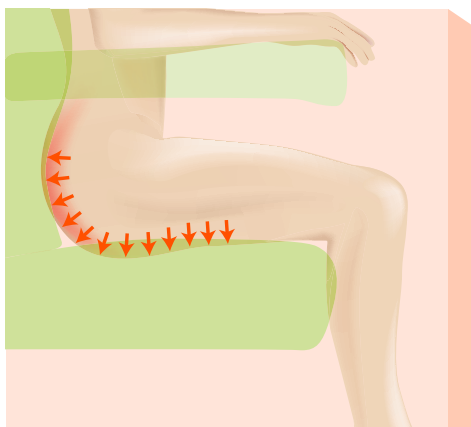

PRESSURE

Key thoughts:

- Selection of pressure cushions is a clinical decision which involves many variables including a risk assessment tool. Professional input from a Tissue Viability Nurse or District Nurse may be required at this stage.
- Vapour permeable fabric as standard on seat base, legrest, backrest and armrests.

PRESSURE CARE OPTIONS


Description	Long Description	Part Number		Select
Visco (std)	Prevention: Heat sensitive memory foam moulds to the user providing maximum support and pressure reduction. Suitable for users who are at risk of pressure damage. 160kg (25 stone) weight capacity.	N/A – standard part		☐
Eлата Alternating Air Cushion - E600C	Management: powered air cushion for users who have developed pressure damage. Weight capacity: 160 kg.	Fits all sizes E600C		☐
Integrated Self Supply	Allows integration of self-supply cushion of approximate dimensions 18" x 18" x 3". (Professional input required. See page 11 for instructions).	N/A – standard part		☐

Cushionair:




The Cushionair cushion is a dynamic pressure relieving cushion which is normally recommended for users who have developed pressure damage*.


The principle is that the cells of the cushion inflate and deflate alternately. This provides pressure redistribution via cyclic changes in loading and unloading (http://www.npuap.org/wpcontent/uploads/2012/03/NPUAP_S31_TD.pdf).

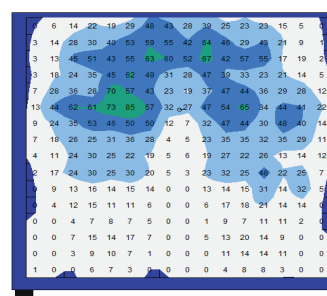


*This is based around anecdotal evidence and Accord's experience; we do not yet have clinical studies to evidence this although these are in progress.

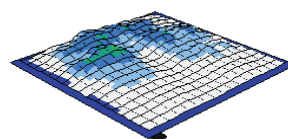

Minimum (mmHg)	0.00
Maximum (mmHg)	84.57
Average (mmHg)	21.51
Variance (mmHg²)	347.67
Standard deviation (mmHg)	18.65
Coefficient of variation (%)	86.69
Horizontal center (cm)	21.73
Vertical center (cm)	28.02
Sensing area (cm²)	1866.24
Regional distribution (%)	100.00

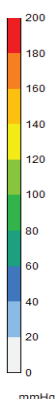

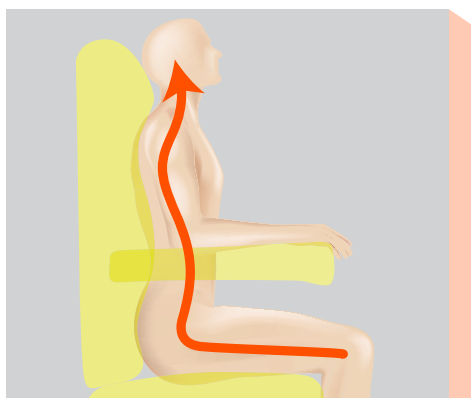

POSTURE

Key thoughts:

- Tilt-in-space can significantly improve midline sitting so increased lateral support may not be necessary if the user tends to slump mildly to one side.
- The Multi-adjustable Pillow Backrest is more adjustable than the Lateral Support Backrest and Adjustable Lateral Support System because filling can be removed to accommodate different postural requirements.

POSTURE MANAGEMENT OPTIONS

Description	Long Description	Part Number		Select
Multi-adjustable Pillow Backrest (std)	Accommodation: accommodates posture that has already developed i.e. kyphosis; our most popular backrest.	N/A Standard Part *NOT COMPATIBLE W/ XL		☐
Lateral Supports	Accommodation: used under the Multi-adjustable Pillow Backrest cushions to provide moderate postural support SOLD IN A PAIR	CA2413		QTY ☐ 2
Cocoon Backrest	Support: offering additional comfort and support, adjusted by changing the position or amount of stuffing in any of its 9 cells. The "reduced shear backrest" mechanism on the Comfort chair complements this backrest by providing extra trunk support, whilst retaining comfort and pelvic position.	Fits all sizes Beige - CA2713 *NOT COMPATIBLE W/ XL		☐
		Black - CA2711 *NOT COMPATIBLE W/ XL		☐
Lateral Support Backrest	Encouragement: encourages a good midline sitting position for users whose posture is flexible.	Beige Small - CA2406 Beige Medium - CA2407 Beige Large - CA2408		S ☐ M ☐ L ☐
		Small - CA2540 Medium - CA2541 Large, XL - CA2542		S ☐ M ☐ L ☐
Adjustable Lateral Support Backrest	Management/correction: contoured foam backrest with adjustable Velcro Lateral Supports. Provides more aggressive lateral support and mild positioning/correction. Supplied with breathable, pressure reducing fabric.	Small - CA2410 Medium - CA2411 Large - CA2412 *NOT COMPATIBLE W/ XL		S ☐ M ☐ L ☐




Profiled Headrest	Suitable for users with reduced lateral head control. Fits over the shoulders.	Fits all sizes Beige - CA2419		☐
		Black - CA2530		☐
2-Button Handset	Suitable for Cognitive Impaired person e.g. dementia (Chair comes Standard w/ 4-button Handset).	Fits all Chair Sizes SC2316 *NOT COMPATIBLE W/ XL		☐
Legrest Channel	Provides side support on legrest to reduce abduction	Small - CA2520 Medium - CA2521 Large - CA2522		S ☐ M ☐ L ☐

Description	Long description	Part Number		Select
Right-hand Side Configura Handset Holder	A great addition for users with dementia, or similar conditions, who may have difficulties locating the handset. Left or right-hand side option. Ideal for users with limited movement on one side of their body from conditions like brain injury or stroke. Users can easily reach the handset from any position, including recline and tilt-in-space, while maintaining optimal posture.	CA2556 *NOT COMPATIBLE W/ XL		☐
Left-hand Side Configura Handset Holder		CA2554 *NOT COMPATIBLE W/ XL		☐

TRANSFER OPTIONS

Description	Long Description		
Drop-down Armside Left-hand side (as sitting in the chair)	Transfers; Enables users to do a slideboard transfer in and out of the chair with ease. Can be bolted on at any time if the users needs change.	Small Black Vinyl - CA2534	☐
		Small Duratek - CA2484	☐
		Small Beige Vinyl - CA2492	☐
		Medium/Large Beige Vinyl - CA2496	☐
		Medium/Large Black Vinyl - CA2535	☐
Drop-down Armside Right-hand side (as sitting in the chair)	Transfers; Enables users to do a slideboard transfer in and out of the chair with ease. Can be bolted on at any time if the users needs change.	Small Black Vinyl - CA2536	☐
		Small Duratek - CA2486	☐
		Small Beige Vinyl - CA2494	☐
		Medium/Large Beige Vinyl - CA2498	☐
		Medium/Large Black Vinyl - CA2537	☐

EXTRAS

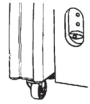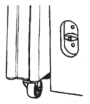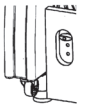Description	Long description	Part Number		Select
Armrest overhang on outside (std)	Provides a good grip to assist the user rolling from side to side for easier fitting of slings.	Standard Part		☐
Armrest overhang on inside	#6 Reduces width between armrests by approximately 38 mm (1.5"). Provides extra arm and elbow support if the user is 'pear shaped' or very thin.	Standard Part		☐
Right hand control (std)	N/A	Standard Part		☐
Left hand control	N/A	Standard Part		☐
Carer control	Fitted to backrest out of user reach	Standard Part		☐
Rise (std)	23° of tilt. The most frequently used rise setting. This provides a gentle, progressive movement.	Standard Part		☐
Medium rise	14° of tilt. There is not a specific user group for this setting – it is normally used if 'Rise' or 'Flat Rise' is inappropriate.	Standard Part		☐
Flat rise	2° of tilt and 254 mm (10") of rise, used for users with Muscular Dystrophy who need to lock knees before transferring. (Not available on Small chair)	Standard Part		☐
Legrest Setting 1.	Standard Position	Standard Part		☐
Legrest Setting 2.	Standard Position + 25 mm (1")	Standard Part		☐
Legrest Setting 3.	Standard Position + 50 mm (2")	Standard Part		☐
Legrest Setting 4.	Standard Position -25mm (1")	Standard Part (rotate the footboard 180 degrees)		☐

TECHNICAL SPECIFICATION

- To provide comfortable seating
- To raise a user from a sitting to a standing position
- To lower a user from a standing to a sitting position
- To tilt in space a user

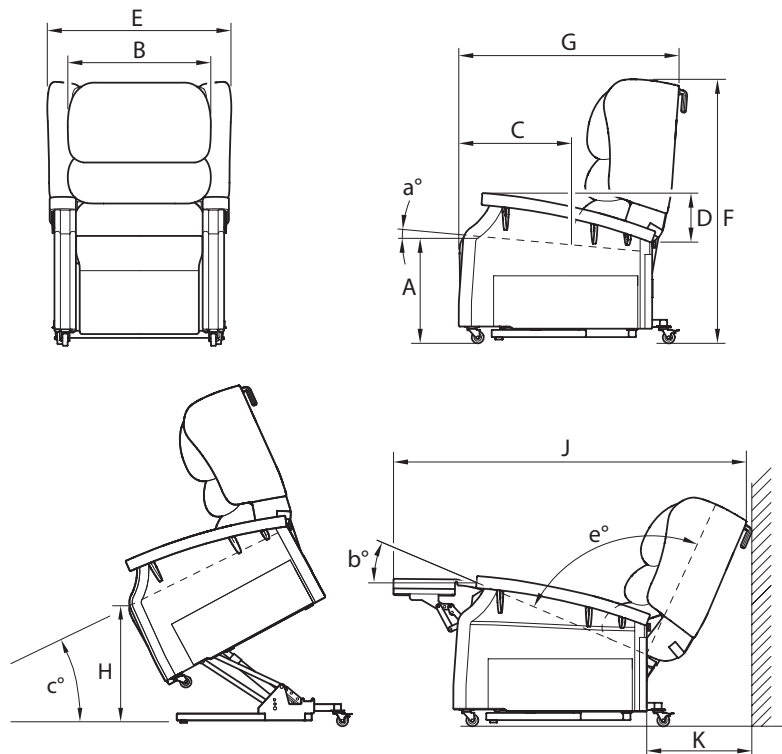


Figure 1: Configura Comfort Dimensions

DIMENSIONS

Dimensions (See Fig 1)		Small	Medium	Large	X-Large 24"
Seat height (A)		406 mm (16")	457 mm (18")	457 mm (18")	400 mm (16")
Seat height with 1" HAKs (A)		431 mm (17")	482 mm (19")	482 mm (19")	425 mm (17")
Seat height with 2" HAKs (A)		457 mm (18")	508 mm (20")	508 mm (20")	450 mm (18")
Seat width (B)		457 mm (18")	508 mm (20")	559 mm (22")	600 mm (24")
Seat depth position 1 (C)		457 mm (18")	457 mm (18")	457 mm (18")	450 mm (18")
Seat depth position 2 (C)		508 mm (20")	508 mm (20")	508 mm (20")	500 mm (20")
Armrest height (D)		180 mm (7.1")	180 mm (7.1")	180 mm (7.1")	180 mm (7.1")
Overall Width (E)		750 mm (29.5")	800 mm (31.5")	850 mm (33.5")	900 mm (27.5")
Overall Height - Upright (F)		1080 mm (42.5")	1115 mm (43.9")	1115 mm (43.9")	1080 mm (42.5")
Overall Length - Upright (G)		885 mm (34.8")	940 mm (37")	940 mm (37")	885 mm (34.8")
Rise height to front of chair (H)	Standard	510 mm (20.1")	545 mm (21.5")	545 mm (21.5")	510 mm (20.1")
	Medium	570 mm (22.4")	620 mm (24.4")	620 mm (24.4")	
	Flat	660 mm (26")	710 mm (28")	710 mm (28")	
Total Length Fully Reclined (J)		1565 mm (61.6")	1660 mm (65.4")	1660 mm (65.4")	1565 mm (61.6")
Chair to wall / furniture (K)		570 mm (22.4")	605 mm (2")	605 mm (2")	570 mm (22.4")

CLEANING INSTRUCTIONS FOR THE FABRIC

Clean and care for the fabric on a regular basis and it will maintain its condition and appearance.

The new generation of Microfibre cleaning cloths without soap are extremely effective for general care and maintenance. You can also wipe clean regularly with a damp cloth to remove dust particles and light soiling. It is recommended to vacuum the upholstery once a week.

Most stains can be removed by wiping clean. Remove spillages promptly (within 15 minutes) with cold water. For extreme soiling control use a 5% Sodium Hypochlorite (Bleach) solution.

Do not machine wash or dry clean.

Do not apply other chemicals such as polish, solvents or stain remover.

INTEGRATED SELF-SUPPLY OVERLAY

The integrated Self Supply Overlay comes as standard on all chairs.

This allows for easy integration of existing cushions.

The size of the aperture is 445 mm x 445 mm x 76 mm (17.5" x 17.5" x 3").

This means the height of the seat and armrests remain unchanged when using another cushion.




STEP 1

Remove the backrest cushions.



STEP 2

Unzip the cushion cover to expose the foam.

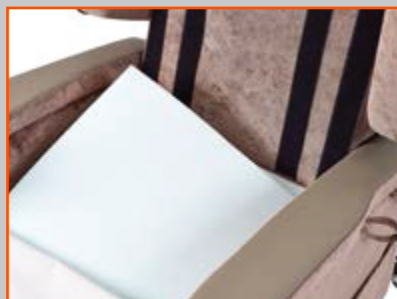

STEP 3

Pull the foam out carefully, starting at the rear and cut the perforations at the front.



STEP 4

This leaves the cut out for the existing cushions to be inserted. Insert the cushion and re zip the cover.



At Enable we source quality, affordable and innovative products globally and have them available for fast delivery. But we do more than just provide products.

We give the knowledge and support to help our customers grow their business and deliver better life care.



ENABLE LIFECARE PTY LTD

T 1300 370 370

enablelifecare.com.au