

Wireless Bluetooth Attendant Controller User Guide



S347429 - Replacement Accessory - Royale Air Carbon Fibre Power Chair Standard Wireless Bluetooth Attendant Controller And Bracket

Contents: 1 x Wireless Bluetooth Controller
1 x Bracket Attachment For The Handles
2 x AAA Batteries

To connect the wireless Bluetooth controller

Step 1 - Install the 2 AAA batteries in the battery compartment located underneath the attendant controller.

Step 2 - Ensure that the powerchair and attendant controller both are turned on.

Note: If there are other powerchairs nearby please turn them Off. Having multiple powerchairs at once may interfere with the Bluetooth signal and hinder successful pairing.

Step 3 - When the Bluetooth controller and power chair are turned on and placed close to each other it should automatically pair.

If failed, please clear the pairing record then repeat this process again.

Clearing the pairing record

Step 1 - Ensure attendant controller is turned off.

Step 2 - Press the Speed Down button and Power Button at the same time and hold for 3-4 seconds until the LED flashes red/green and then turns off.

Step 4 - The Pairing record is now cleared, and the remote will no longer auto-connect to the previous controller.

If unsuccessful, repeat this process again.